

Redmon Precision Scale

Instruction Manual for Item #7475 Large Pet Scale

Features:

Large Platform 16" x 28"	Easy to read display
Maximum weight capacity 225 lb/ 102 kg	Minimum weight capacity 1 lb 2 oz / .5 kg
Low battery indicator "Lo"	Over weight indicator "Err"
Tare Function	Hold Function
Unit of measure in pounds or kilograms	Graduation: 2 oz/50 g increments
Power: Two AAA alkaline batteries (Not included)	

Preparations:

1. Battery Installation: Remove the battery compartment cover located on the bottom side of the scale. Insert two AAA alkaline batteries, using the diagram inside the compartment for the + and -.
2. Place the scale on a flat, level, hard surface making sure all four sensors/feet are in contact evenly with the surface. Do not attempt to weigh if the scale teeters or rocks; you must find a flat, hard surface. Do not attempt to weigh on carpet or a soft surface. Never leave a pet unattended on the scale.
3. Press the On/Off button. The digital display will activate and show the weight as zero and the unit of measure to be used.
4. To change the unit of measure you wish to use, pounds/ ounces or kilograms, press the Unit button and note the unit of measure displayed on the display screen.



Weighing Your Pet:

1. Press the On/Off button, 0: 0 will appear on the display screen. Place your pet on the scale platform near the center or evenly distributed. When the pet is relatively still, press the Hold button. (Note: you must press the Hold button for the scale to calculate and record the weight in the display screen) The display screen numbers will flash several times while determining the correct weight and then automatically display the weight for about six seconds before turning off.

Tare Function:

1. The Tare Function allows you to place an object such as a cage or bed on the scale and still record only the weight of the pet.
2. Place the object on the scale and then press the On/Off button. The scale will process the added weight and display 0: 0. Place the pet on the scale and only the weight of the pet will be calculated and displayed.

Care:

1. Use and store in a dry area only. Do not leave loads on the scale.
2. Do not expose to extreme temperatures and/or high humidity. Best operating temperatures are 65-85 degrees Fahrenheit.
3. Do not overload the scale beyond the maximum weight of 225 pounds.
4. Clean the platform with a damp cloth and wipe dry. Do not allow water or moisture to enter the inner workings. This scale is not waterproof.

Warning: Never leave your pet unattended during the weighing process.

Disposal: Once the product life has ended disposal should be carried out according to current regulations. Used batteries must be disposed of in special recycling bins.



Distributed by W. C. Redmon Co., Inc.
Established 1883 – Peru, Indiana U. S. A.

www.RedmonUSA.com

Made in China